

BETADINE®

COLD DEFENSE nasal spray

Medical Device

Introduction

The most common route of infection of cold is through the nose. Cold viruses entering the nose attach to the nasal lining where they multiply and spread. This then leads to the symptoms such as stuffy, runny nose, sore throat, coughing, fever, aches and pains and fatigue.

What is BETADINE® COLD DEFENSE Nasal Spray?

Powered by a natural ingredient Carragelose®, **BETADINE® COLD DEFENSE Nasal Spray** is an easy to use nasal spray clinically proven to fight against cold viruses. If used at the first sign of cold, **BETADINE® COLD DEFENSE Nasal Spray** shortens the duration of cold symptoms by 2 days. It also moistens the nasal lining to protect it from dryness and its unfavorable effects.

How Does BETADINE® COLD DEFENSE Nasal Spray work? **BETADINE® COLD DEFENSE Nasal Spray** targets the area in the nasal cavity where cold viruses first take hold and multiply.

It forms a moisturising protective film on the nasal mucous membrane. This liquid layer serves as a physical barrier against external influences such as cold viruses, air pollution and low air humidity.

Its natural ingredient Carragelose® physically traps the viruses on the nasal lining, hinders their spread and multiplication, as well as helps your body to remove them naturally.

BETADINE® COLD DEFENSE Nasal Spray is Preservative free and Sterile.

BETADINE® COLD DEFENSE Nasal Spray is proven to eliminate 99% of cold causing viruses.

BETADINE® COLD DEFENSE Nasal Spray is suitable for adults. Can be used during pregnancy or lactation under the advice of your doctor.

It is non-drowsy and non-addictive.

Direction for Use

1 spray into each nostril 3 times a day. Treatment should begin as early as possible after the first sign of cold.

1. Remove the protection cap before use and pump a few times if necessary until spray is discharged. **BETADINE® COLD DEFENSE Nasal Spray** is now ready for use.

2. Spray one time into each nostril. As far as possible, hold the bottle upright when spraying. Replace protective cap after use.

Special precautions for safe use

Do not use if allergic to any of the ingredients.

In case of adverse reactions, stop treatment and consult a doctor or pharmacist.

Keep out of reach of children.

To avoid contamination do not use this product for more than one person.

Avoid contact with eyes.

Do not refrigerate. Do not freeze. Store below 30°C. Protect from light.

Do not use after the expiration date indicated on the carton and bottle label has passed.

Ingredients

Salt solution with 1.2mg/mL Carragelose® (Carrageenan)

Manufactured by:
Laboratoires Pharmaster
Z.I. de Krafft
67150 ERSTEIN – France

Imported and distributed by:
Mundipharma Distribution GmbH
(Philippine Branch)
Unit 3701-3708 High St. South
Corporate Plaza, Tower 2,
26th St. Cor. 11th Ave.,
Bonifacio Global City, Taguig,
Metro Manila

MADE IN EUROPE

CMDN NO: CDRRRH-2021-748418

®: BETADINE is a Registered Trademark of Mundipharma.

®: CARRAGELOSE is a Registered Trademark of Marinomed Biotech AG.

RM-BET-1191-V2-0716
MO NSAD.20M PL.PH.02

NOT37916V01